MaxiAspirator™

Aspirator System

601

Patient Manual

CAUTION: USA Federal law restricts this device to sale by or on the order of a physician.

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MaxiAspirator™

Table of Contents

Specifications	2
Introduction	3
Important Parts of the MaxiAspirator	4
Setup and Operating Instructions	5
Cleaning	6
Maintenance	6
Troubleshooting	8
Warranty Information	9
Important Safety Information	10

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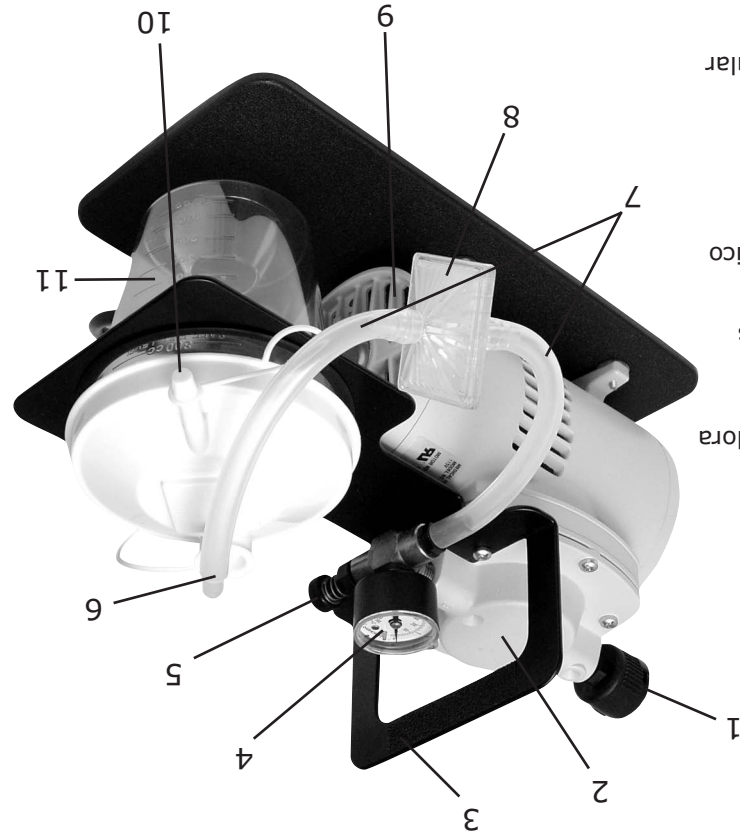
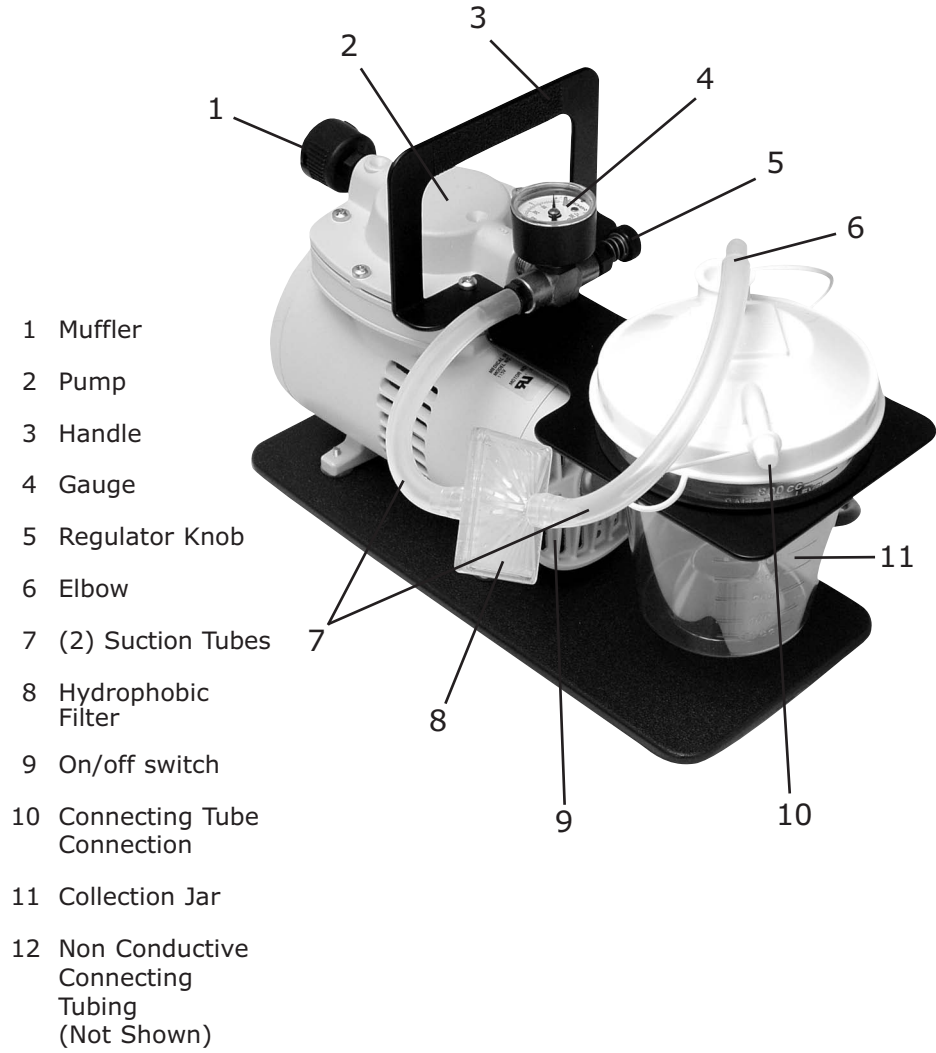
The MaxiAspirator device, which was created for institutional or long term home care use, is a very reliable and a consistent performer. The MaxiAspirator is a high performance compressor with 23 inches Hg maximum vacuum, strong flows resulting in shorter recovery times, precision regulator and gauge, and non-rusting base and components. The handle will accommodate either 800cc or 1200cc jars.

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Important Parts of the MaxiAspirator

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1. Make sure the power cord is unplugged and the power switch is in the OFF position.
2. Inspect the unit, including the power cord, for any damage. Contact Care Provider to have any damaged parts replaced.
3. Set the MaxiAspirator on a level, sturdy surface so the controls of the unit can be easily reached and adjusted.
4. Connect suction tubing, elbow and filter assembly to the outlet on the regulator and inlet on the collection jar as shown on page 4. Make sure all tubing connections and the collection jar lid are pushed on tight to ensure a proper vacuum level is achieved.
5. Connect the AC power cord located on the back of the unit to a standard 110V AC wall outlet
6. Turn the switch to the ON position.
7. Set suction level by simultaneously occluding the outlet of the collection jar and turning the regulator knob until the required suction registers on the gauge. Turn the regulator clockwise to increase vacuum and counter-clockwise to decrease vacuum. Observe the vacuum level on the unit's gauge.
8. Attach the non conductive connecting tubing to the outlet on the collection jar.
9. Attach your catheter (not included) to the connecting tubing.
10. Proceed with the treatment.
11. At the conclusion of the treatment, turn the switch to the OFF position.

NOTE

The in-line hydrophobic filter is designed to prevent foreign material from being drawn into the compressor. At a minimum, change the filter whenever foreign material is drawn into the tubing.

NOTE

Do not move MaxiAspirator with a full collection jar. Liquids may flow back through the device resulting in damage to the internal components of the compressor.

NOTE

Cold temperatures may result in poor compressor performance. Bring machine up to room temperature for best results.

MaxiAspirator Cleaning

CAUTION

Never submerge the MaxiAspirator in water, hold under running water or spray with water or other liquids. This will result in compressor damage and could result in electrical shock.

1. Make sure the ON/OFF switch is in the OFF position.
2. Unplug the MaxiAspirator from the wall outlet.
3. Wipe MaxiAspirator with a clean, damp cloth every 2-3 days. This will keep the unit dirt and dust free.

TUBING AND COLLECTION JAR

1. Verify the device is turned OFF and allow the suction to drop off.
2. Disconnect the tubing from the collection jar and from the device.
3. Rinse the tubing and collection jar thoroughly with three parts hot tap water and one part vinegar.
4. Rinse the parts with hot tap water and allow them to air dry.
5. Reconnect the tubing and hydrophobic filter between the collection jar and the device.

Maintenance

WARNING

Do not remove compressor case. Failure to comply could result in electrical shock. All disassembly must be carried out by an authorized EVO Medical Solutions service center.

All major maintenance on the MaxiAspirator MODEL #601 must be performed by an authorized EVO Medical Solutions service center.

The MaxiAspirator is equipped with a muffler (used to dampen the noise from the compressor) and a hydrophobic filter (used to prevent back-up of fluid into the unit).

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HYDROPHOBIC FILTER REPLACEMENT

1. Disconnect the tubing from both ends of the filter. Discard the old filter.
2. Attach the new filter so the tubing from the suction jar is connected to the 'fluid' side of the filter. Connect the tubing from the pump to the opposite side of the filter.

SUCTION TUBING/COLLECTION JAR/HYDROPHOBIC FILTER

The suction tubing, collection jar and hydrophobic filter supplied with the MaxiAspirator are recommended for single patient use and should be replaced at a minimum between patients and as per physician order. If the filter gets wet, it must be replaced.

MUFFLER REPLACEMENT

The muffler should be replaced every three months or more often if contaminated.

1. Grasp the muffler housing and unscrew it from the pump by turning it counter-clockwise.
2. Screw in the replacement muffler (Re-order No. 612).

WARNING

To reduce the risk of electric shock, always unplug the power cord from the power source before servicing.

NOTE

Should fluid be aspirated back into the unit, dealer service is necessary as possible compressor damage may result.

NOTE

Operating the MaxiAspirator without the hydrophobic filter may allow fluids to enter the compressor causing damage to its internal components.

Troubleshooting

Symptoms	Possible Causes	Remedies
Reduced/low vacuum	<ol style="list-style-type: none"> 1. Clogged hydrophobic filter or tubing. 2. Regulator improperly set. 3. Air leak. 	<ol style="list-style-type: none"> 1. Replace the filter or tubing following directions in this manual. 2. Set regulator following operating instructions. 3. Examine the tubing and connectors for possible leaks. Attach, tighten, or replace tubing as necessary. Make sure the jar lid is on tightly as well.
Compressor does not start	<ol style="list-style-type: none"> 1. Power cord not plugged in. 2. Defective switch. 3. Backed up pressure in the compressor. 	<ol style="list-style-type: none"> 1. Plug in the power cord. 2. Contact your dealer or the EVO service center. 3. Disconnect tubing from the collection jar and turn on the device.
Gauge reads improperly	Faulty gauge.	Replace gauge.
Compressor runs but gauge reads zero	Gauge reads line vacuum.	Connect tubing to the suction jar and adjust the regulator.

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evo Medical Solutions warrants the MODEL #601 MaxiAspirator for 1 year from the date of purchase due to faulty parts or workmanship. Check with your dealer for more details. This warranty is limited to the dealer. Any defective part(s) or assembly will be repaired or replaced, at the sole discretion and determination of evo Medical Solutions if the unit has not been misused or tampered with during the warranty period. Normal maintenance items, as outlined in this manual and disposable components are not covered by this warranty. Shipping charges, if any, shall be paid by the purchaser.

NOTE

There is no other express warranty. Implied warranties, including those of merchantability and fitness for a particular purpose, are limited to the duration of the express limited warranty and to the extent permitted by law and all implied warranties are excluded. This warranty does not cover providing a loaner compressor, compensating for costs incurred for compressor rental, or for labor costs incurred in repairing or replacing a defective part(s).

Important Safety Information

This eVO Medical Solutions product is an electrical device. Always follow basic safety precautions when using this device including but not limited to the following:

READ THIS SAFETY CARD BEFORE USING

WARNING: To reduce the risk of burns, fire, electrocution, or other bodily harm or injury to persons:

1. Always unplug this device after each use.
2. Never submerge this product in water.
3. Never use this product while bathing, showering, washing dishes, or around water source of any kind.
4. Never leave this product unattended.
5. Use this product only for its intended use as described in the operator manual.
6. Closely monitor and supervise use by invalids and children.
7. Never block the air vent openings or allow objects to fall or to be inserted into the air vent openings.
8. Observe caution when using this product where oxygen is being administered.
9. If this product is being used in a life threatening situation, a backup unit is necessary.

SAVE THIS SAFETY CARD

WARNING: This equipment is not suitable for use in the presence of a flammable anaesthetic mixture with air or with oxygen or nitrous oxide.

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