# Reduced Gap Deluxe Full-Length Bed Rail

## Assembly, Installation and Operating Instructions

Model Nos. 6628 and 6628-TSS (extra-wide crossbraces)

Yes, you can.™

### SAVE THESE INSTRUCTIONS

*NOTE: Check ALL parts for shipping damage before using. In case of damage, DO NOT use the equipment. Contact the Dealer/Carrier for further instructions.*

*NOTE: A brochure entitled "A Guide to Bed Safety; Bed Rails in Hospitals, Nursing Homes and Home Health Care: The Facts" can be found on the FDA's website at <http://www.fda.gov/cdrh/beds/>.*

### SAFETY SUMMARY

To ensure the safe use of the Reduced Gap Deluxe Full-Length Bed Rail, these instructions **MUST** be followed:

#### GENERAL WARNINGS

**DO NOT** install or use this equipment without first reading and understanding these instructions. If you are unable to understand the **Warnings, Cautions or Instructions**, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment - otherwise, injury or damage may occur.

When used with a manual/electrical bed, the bed rails **DO NOT** fall under any weight limitations. Bed rails can be deformed or broken if excessive side pressure is exerted on the bed rails. These bed rails are used for the purpose of preventing an individual from inadvertently rolling out of bed. The bed rails are **NOT** intended nor may be used for restraint purposes. If an individual is capable of injuring himself/herself, a physician or a healthcare professional should be consulted for alternative means of safe restraint.

Replacement mattresses and bed rails with dimensions **DIFFERENT** than the original equipment supplied or specified by the bed manufacturer may not be inter-changeable. Variations in bed side rail design, thickness, size or density of the mattress could cause entrapment. Use **ONLY** authorized Invacare replacement parts.

**DO NOT** use the bed rails as push handles when moving the bed.

This bed rail is **NOT** a handhold for getting into or out of bed.

#### INSTALLATION AND OPERATION WARNINGS

After **ANY** adjustments, repair or service and **BEFORE** use, make sure all attaching hardware is tightened securely.

Always test to make sure that the bed rails are properly and securely in place **BEFORE** using.

One (1) crossbrace **MUST** be positioned between the **FIFTH** and **SIXTH** springs at the **HEAD-END** of the bed. The remaining crossbrace **MUST** be positioned between the **SECOND** and **THIRD** springs at the **FOOT-END** of the bed. Otherwise, either the bed rail or the bed may be damaged when the head end and/or foot end are raised.

To ensure proper bed rail installation, arrows on crossbrace decal **MUST** point to center of bed.

The **OUTER TELESCOPING TUBE** of the rail **MUST** be positioned at the **HEAD-END** of the bed to prevent a pinch point when raising/lowering bed.

Install fully extended rails on bed. Full length side rails are not intended for use as half rails.

## INSTALLATION AND OPERATION WARNINGS (CONTINUED)

Make certain that the distance between the rails does not exceed width of mattress. Otherwise, entrapment may occur between bed rail and mattress or bed frame.

Raise the bed rail to maintain the necessary height of the side rail for patient safety keeping in mind the gap between the bottom of the side rail and the mattress.

After raising/lowering the HEAD/FOOT END of the bed, check the gap between the bottom of the bed rail and the mattress. Adjust the height of the bed rail or provide alternative means of patient protection. Otherwise, entrapment may occur between bed rail and mattress or bed frame.

Unless the rail is in the lowest position, ensure that the spring loaded knobs fully protrude through the same respective adjustment hole of each rail tube. This ensures that the rail is securely locked in position and an even height adjustment is achieved.

## INSTALLATION

*NOTE: Refer to INSTALLATION AND OPERATION WARNINGS in the SAFETY SUMMARY of this instruction sheet.*

### INSTALLING THE CROSSBRACES (FIGURES 1 AND 2)

1. Remove the mattress from the bed.
2. Place the bed in the flat position and position one (1) crossbrace between the FIFTH AND SIXTH springs at the HEAD-END of the bed frame as shown in FIGURE 1.

*NOTE: If the bed frame contains notches between the FIFTH AND SIXTH springs at the HEAD-END of the bed, position the crossbrace in the notches.*


3. When holding either end of the crossbrace assembly, turn the barrels so plunger knobs are on top as shown in FIGURE 1.

*NOTE: Ensure the long end of the barrel faces the center of the bed with the decal to the center of bed as shown in FIGURE 1. If long end faces the opposite way, rotate crossbrace 180°.*

4. Position the bed hooks so they are on the bottom side of the crossbrace as shown in FIGURE 2.
5. Perform one (1) of the following:
  - A. STANDARD HARDWARE INSTALLATION - Perform the following steps:
    - a. Attach one (1) bed hook onto the bed spring angle iron and push in on the spring loaded bed rail until you are able to attach the opposite end to the bed spring angle iron as shown in FIGURE 2.
    - b. Rotate the barrel so that the long ends face down.

*NOTE: Ensure plunger pins are facing the center of the bed.*

- B. OPTIONAL HARDWARE INSTALLATION - Perform the following steps:
  - a. Attach wingnuts to bed hooks.
  - b. Tighten until the wingnut reaches the bed spring angle iron.
  - c. Rotate the barrel so that the long ends face down.

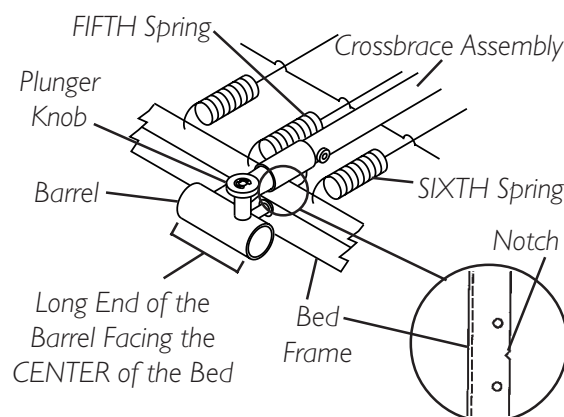
**FIGURE 1 - INSTALLING THE CROSSBRACES - BARRELS**

6. Visually inspect crossbrace to confirm proper orientation.
7. Position the remaining crossbrace between the **SECOND** and **THIRD** springs at the **FOOT-END** of the bed. Repeat **STEPS 3-6** for the **FOOT-END** crossbrace.

*NOTE: If the bed frame contains notches between the **SECOND** and **THIRD** springs at the **FOOT-END** of the bed, position the crossbrace in the notches.*

8. Place mattress on bed frame.
9. Install the side rails. Refer to **INSTALLING THE SIDE RAILS** in this instruction sheet.

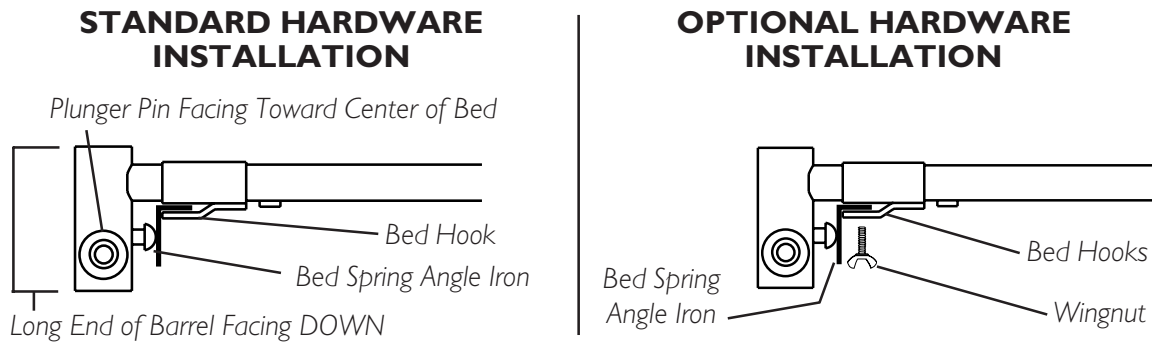
**FIGURE 2 - INSTALLING THE CROSSBRACES**

### **INSTALLING THE SIDE RAILS (FIGURE 3)**

*NOTE: Ensure crossbraces are 41 to 68-inches apart.*

1. Extend one (1) side rail to reach the **HEAD-END** and **FOOT-END** crossbraces.
2. Insert the **OUTER** side rail into the barrel of the **HEAD-END** crossbrace.
3. Pull and hold the plunger pin on the **HEAD-END** barrel while lowering the side rail.
4. Release the plunger pin on the **HEAD-END** barrel.
5. Raise or lower the bed rail until the plunger knob "clicks" securely into the adjustment hole or until the rail stop securely rests against the barrel.

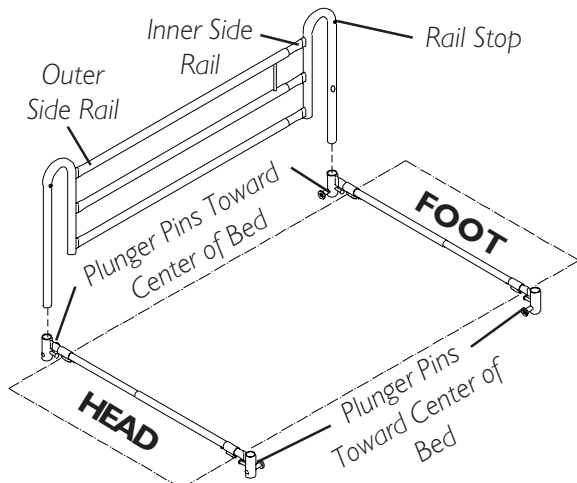**FIGURE 3- INSTALLING THE SIDE RAILS**

6. Insert the **INNER** side rail into the barrel of the **FOOT-END** crossbrace.
7. Repeat **STEPS 3-5** for the **INNER** side rail and the **FOOT-END** crossbrace, adjusting the bed rail to the position reached in **STEP 5**.
8. Repeat **STEPS 1-7** to install the other bed rail onto the opposite side of the bed.

### **LOWERING/RAISING THE SIDE RAILS**

*NOTE: Refer to **INSTALLATION AND OPERATION WARNINGS** in the **SAFETY SUMMARY** of this instruction sheet.*

1. Hold rail, while pulling the plunger knob.
2. Lower/Raise rail.
3. Release plunger knob and lower/raise the rail until plunger knob "clicks" into position or until the rail stop securely rests against the barrel.
4. Repeat **STEP 3** for opposite end of bed rail.
5. Ensure **HEAD-END** and **FOOT-END** of bed rails are adjusted to the same height.

---

## **LIMITED WARRANTY**

**PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.**

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants its product to be free from defects in materials and workmanship for three (3) years for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address below. Provide dealer's name, address, model number, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. **DO NOT** return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

**LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN AUTHORIZED INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.**

**THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.**

**THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/ PROVINCIAL LAWS AND REQUIREMENTS.**

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© 2005 Invacare Corporation
Invacare is a registered trademark of
Yes, you can. is a trademark of
**Invacare Corporation** [www.invacare.com](http://www.invacare.com)