
Classification and Symbol Glossary:

-  Attention, consult accompanying documents
-  Class II equipment (Double Insulated)
- IPX1 Drip proof equipment (for the 230V unit only)
-  ON (power)
-  OFF (power)
-  Alternating current
-  Type BF applied part

The models 626 and 626-10 are classified as:

- IEC class II
- Type BF applied part
- IPX1 drip proof (for the 230V unit only)
- Continuous operation
- Not suitable for use in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide.

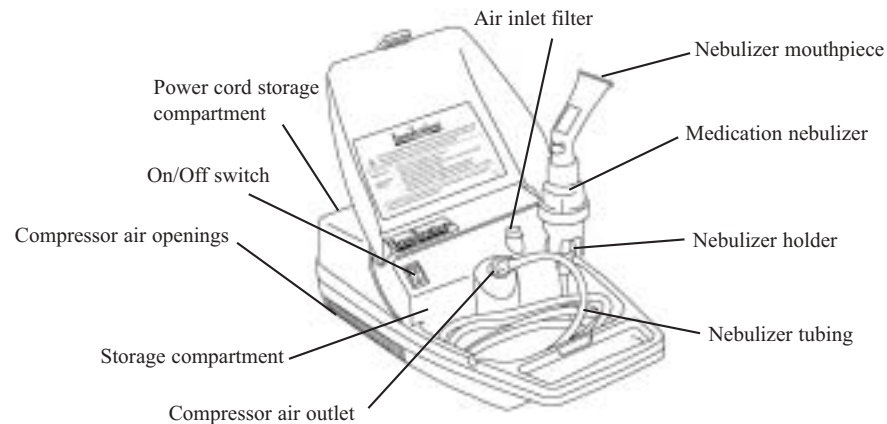To treat your respiratory disorder, your doctor has prescribed a medication that must be delivered in an aerosol form. The medicine, as a mist, can reach deep in your lungs where it is most effective. The Inspiration 626 and 626-10 efficiently direct airflow through the medication nebulizer, where the medication is actually nebulized, for delivery into your lungs.

Operating Instructions:

1. Place the unit on a firm, flat surface free of clutter. Choose a place near an electrical outlet where you will be comfortable during the treatment.
2. Plug the power cord into an AC electrical outlet. Do not use an extension cord or plug the unit into an outlet with other large appliances.
3. Wash your hands before each treatment.
4. Assemble your medication nebulizer as pictured and place the prescribed medication into it.
5. Connect one end of tubing to the compressor air outlet and the other to the bottom of your medication nebulizer. You are now ready to begin your breathing treatment.
6. Turn the unit on to begin treatment.
7. Turn off and unplug the unit when all the medication has been delivered as prescribed.

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician or licensed healthcare professional.

	Model no. 626	Model no. 626-10
	<u>115 V Unit</u>	<u>230 V Unit</u>
Input Voltage:	115 VAC	230 VAC
Input Frequency:	60 Hz	50 Hz
 Storage/transport temperature:		-25°C to 70°C
RH Storage/transport humidity:		10% to 95% Noncondensing
P Storage/transport atmospheric pressure:		690 to 1060 hPa
Operating temperature:		10°C to 40°C
Operating humidity:		15% to 95% Noncondensing

Warranty Information

Respironics, Inc. warrants the Inspiration 626 and 626-10 compressor to be free from defects in materials and workmanship under normal use and operation for a period of 5 years from date of purchase from Respironics, Inc. The warranty is limited to the repair of or replacement, at Respironics, Inc.'s sole option, of any such component or equipment claimed to be defective when claim is shown to be bona fide by evaluation by Respironics, Inc. This warranty does not extend to any component or equipment subjected to misuse, improper operation, accidental damage or unauthorized repairs, and is not extended to charges of, or for, labor repairs. All items returned must be properly packaged and shipped, prepaid, by the home care dealer servicing the unit. Respironics, Inc. shall not be liable to purchaser or others for loss of use of equipment or for indirect, or incidental or consequential damages that might arise.

Respironics New Jersey, Inc.
41 Canfield Road
Cedar Grove, New Jersey
07009 · 1201 USA

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Respironics Asia Pacific
Unit 2301, 23/F Skyline Tower
39 Wang Kwong Road
Kowloon Bay, Kowloon, Hong Kong

Cleaning the medication nebulizer after each treatment:

1. Wash your hands.
2. Remove the tubing from the bottom of the medication nebulizer. This should be done in a gentle twisting motion. There is no need to disinfect the tubing because it makes no contact with the medication.
3. Unscrew the lid from the medication nebulizer cup and remove the centerpiece.
4. Wash the mouthpiece and parts of the medication nebulizer in warm water with a mild dishwashing soap and rinse thoroughly.
5. Then, soak these items in three parts hot water to one part distilled white vinegar (5% acidity) for 30 minutes.
6. Remove items from the vinegar solution, rinse under hot water and allow to air dry. Do not towel dry or wash in a dishwasher.
7. When completely dry, reassemble and place in the storage compartment.
8. Close the lid of the unit when not in use.

Note: The medication nebulizer you received with your Inspiration unit is disposable and should not be used for extended periods of time without replacement. Contact your local home care dealer for replacement guidelines and supplies.

Maintaining the unit:

Before cleaning the unit, it is important that it is unplugged. You may use a damp cloth to wipe down the exterior case of the unit. **Never submerge the unit in water!** If medical disinfectants are used, be sure to follow the manufacturer's instructions. The Inspiration compressor should not be lubricated.

Changing the filter:

The Inspiration filter should be inspected every week for replacement. Operating the unit with a dirty filter can impair the proper operation of the unit-causing defects that are not covered under the warranty.

1. To remove the filter, twist slightly and pull straight up.
2. The filter should be replaced when it becomes dirty and discolored. Do not wash or reuse the filter. The average life of the filter will depend on usage and the operating environment.
3. Contact your local home care dealer for new Respironics, Inc. filters.

Warnings and Cautions:

Failure to read and observe all precautions could result in personal injury or equipment damage.

- Never use this unit if it has a damaged power cord or plug. Never use an extension cord or electrical adapter (cheater plug). Keep cord away from heated surfaces.
- Never block the air openings on either side of unit by operating the unit on a soft surface like a bed or couch. Always operate on a hard surface free of clutter. Keep the air opening free of lint, hair and the like. Never drop or insert objects into any of the air openings.
- Never use while bathing or around water. Do not place or store the unit where it can fall or be pulled into a tub or sink.
- Never place the unit in or drop in water or other liquid. Do not reach for a unit that has fallen into water. Unplug immediately.
- Do not leave unit unattended while plugged in. Always unplug it immediately after use.
- Never disassemble the unit. Service should be performed only by an authorized dealer. Unauthorized repair voids the warranty.
- Never operate the unit where aerosol (spray) products are being used or where oxygen is being administered in a closed environment such as an oxygen tent.
- Children and invalids should always use under strict supervision of an adult who has read this manual.
- The unit should not be adapted or converted to DC current, or used with an electrical input frequency other than the one specified.
- This medical device is only to be used when prescribed by a physician and operated in accordance with this manual.
- The unit should not be used around flammable substances, or in the presence of a flammable anesthetic mixture with air, oxygen or nitrous oxide.
- Never use while sleeping or when drowsy.

Safety Thermal Overload Protection:

The motor has a thermal overload protector. In the event the compressor becomes overheated, it will shut off the motor. If this occurs, turn the unit off and wait at least 20 minutes before trying to resume use. Check to make sure the compressor air openings are not blocked and that the filter is clean. If this is a common occurrence, you may have a low voltage situation and should call your home care dealer.

Electromagnetic Compatibility:

The Inspiration 626 and 626-10 are designed to meet the current electromagnetic compatibility requirements. However, if you suspect operation of this device interferes with the normal operation of your TV, radio or other electronic appliance try 1) relocating the appliance or the unit until the interference stops or 2) plugging the unit into a different power outlet controlled by a separate circuit breaker or fuse.

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