



## Welcome to your React PAP Device

(G3 CPAP, G3X APAP, or G3 25A BPAP)

### Follow These Simple Steps to Get Started

**Step 1:** Download the manufacturer's smartphone app, React Health PLUS, to your mobile device. This app will guide you through the setup process. You can find the app by searching for React Health PLUS on the App Store or scan the QR code provided by React Health.

**Step 2:** For additional therapy guidelines and education videos, go to the URL: [Apria.com/sleep/getting-started](https://Apria.com/sleep/getting-started) on your browser, or scan the QR code on your mobile device.


**Step 3: Schedule set-up call** — A text will be sent to your phone to schedule your call. If you don't have a set-up call scheduled, please contact us at **877-265-2426, option 4**, for assistance.

**Step 4: Stay on top of your PAP therapy with myApria** — Order eligible replacement supplies, update your profile, and manage your contact and insurance information with just a few clicks. Visit [myApria.com](https://myApria.com) or download the mobile app, **available in the App Store and Google Play.**







**1. Unbox Your CPAP Machine:** Remove the machine, power cord, tubing, and mask.

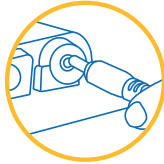

**2. Connect the Power Cord:** Plug the power cord into the back of the machine and into a wall outlet.

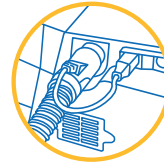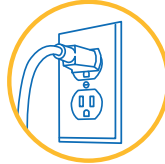

**3. Connect the Tubing:** Attach one end of the tubing to the air outlet on the machine.



**4. Connect the Mask:** Attach the other end of the tubing to your CPAP mask.



**5. Fill the Water Chamber (if applicable):** Open the water chamber and fill it with distilled water to the fill line.



**6. Put On Your Mask:** Place the mask comfortably on your face, ensuring a secure fit.



**7. Turn On the Device:** Press the power button on the machine. There's no need to change any settings—**the device is already set up for your comfort.**

## Need Help?

Sleep Coaches are available for any questions. Call us at:

**877.265.2426 Option 4**