


Invia[®] Liberty[™] Quick Card – Clinician




The Invia Liberty Quick Card does not replace the Invia Liberty instructions for use. For information concerning intended use, indications, contraindications, warnings and cautions please refer to the Invia Liberty Clinician instructions for use Ref 200.1114 available with the Invia Liberty pump or at www.medela.com/healthcare.


Preparation for use

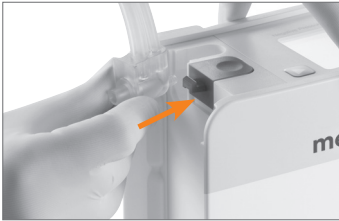


1. Attach Invia Liberty canister tubing to the pump.

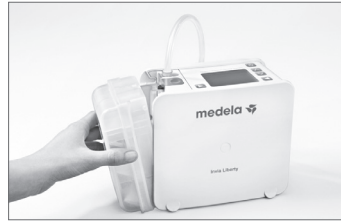

2. Click Invia Liberty canister into place.

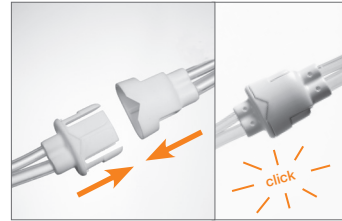


3. Connect canister tubing to the dressing tubing.

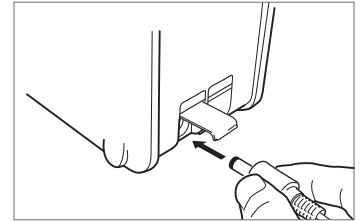


4. Charge battery for the first time before starting therapy. When completely discharged, approximate charging time is 4 hours. Recharge battery throughout the length of therapy. The expected battery time generally exceeds 10 hours.


New patient therapy set up

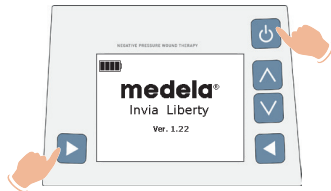


1. Switch pump ON in Administrative mode, press and hold simultaneously.

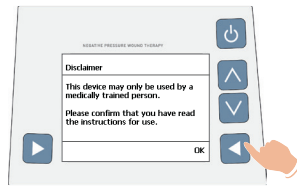

2. Acknowledge disclaimer, confirm with "OK".

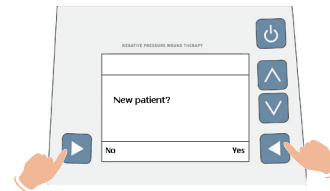

3. Choose "Yes" to set new therapy number or "No" to keep same therapy number.

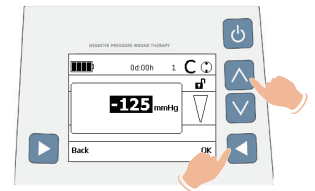


4. Press selection buttons up or down to set pressure level.
5. Press "OK" to confirm and enter main display.


Basic operating commands




On/Off
1. Switch pump ON.
2. Switch pump OFF, press and hold for 3 seconds.

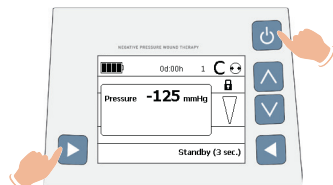

Administrative mode
1. Enter Administrative mode, press and hold simultaneously.

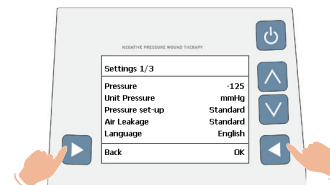


Choose settings
1. Enter settings menu in Administrative mode, press and hold simultaneously.
2. Use selection buttons up or down to navigate.

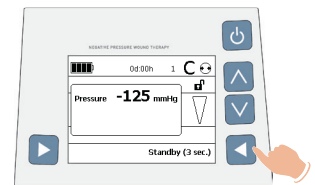


Enter/exit standby
1. Press "Standby" to enter standby mode.
2. Press "ON" to resume therapy.


Change pressure level – Standard mode

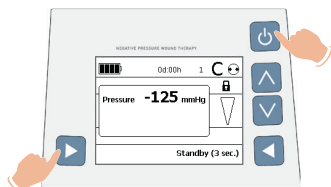


1. Enter Administrative mode, press and hold simultaneously.

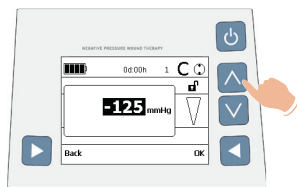

2. Press selection buttons up or down to set pressure level from preset menu. For pressure level outside preset values see detailed mode.
3. Press "OK" to confirm and enter main display.

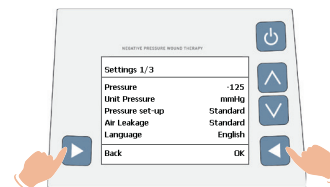


1. Enter settings menu in Administrative mode, press and hold simultaneously.
2. Scroll down with down button and select Pressure set-up.
3. Press "OK" to select.
4. Press selection buttons up or down to choose Detailed mode. Press "OK" to confirm.

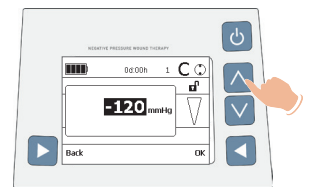

5. To return to the main display, press "Back".
6. Press selection buttons up or down to set pressure level in increments of 5 mmHg.
7. Press "OK" to confirm and enter main display.

Change pressure level – Detailed mode


Change canister

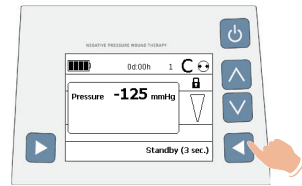


1. Press "Standby" for 3 seconds.
2. Clamp canister and dressing tubing.



3. Release and remove canister (if necessary, replace canister tubing).
4. Click new Invia Liberty canister into place.
5. Unclamp canister and dressing tubing.

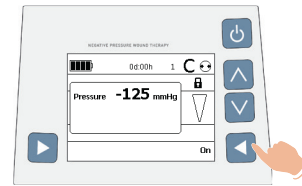


6. Press "On" to resume therapy.

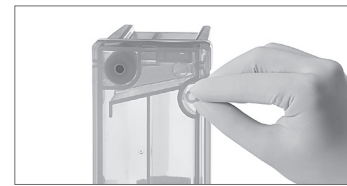


7. Seal used canister with a cap.
8. Dispose used canister in accordance with local procedures.


Select air leakage volume

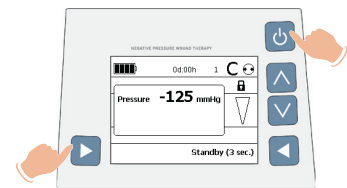


1. Enter Administrative mode, press and hold simultaneously.

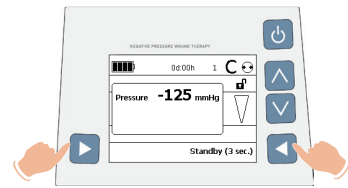

2. Enter settings menu, press and hold simultaneously.

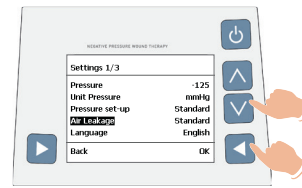


3. Scroll down to "Air Leakage".
4. Press "OK" to confirm.

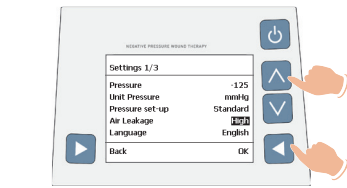


5. Press selection buttons up or down to set air leakage volume "Standard" or "High".
6. Press "OK" to confirm.
7. Press "Back" to return to main menu.


Change constant/intermittent therapy mode

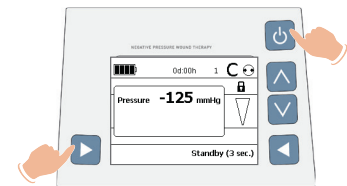


1. Enter Administrative mode, press and hold simultaneously.

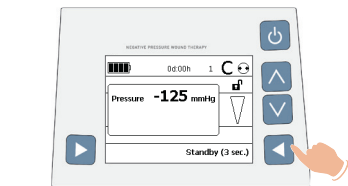

2. Press "Standby" for 3 seconds to put pump in standby mode.

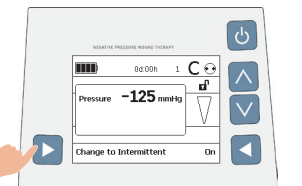


3. Press "Change to Intermittent". Default times for intermittent mode are 5 minutes on and 2 minutes off.

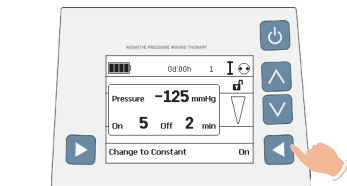


4. Press "On" to begin therapy. To change ON and OFF time in intermittent mode, refer to the instructions for use – Chapter 'Change settings'.


Log file

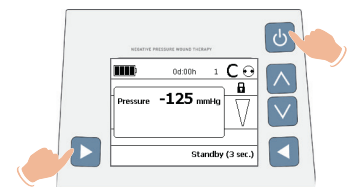


1. Enter Administrative mode, press and hold simultaneously.

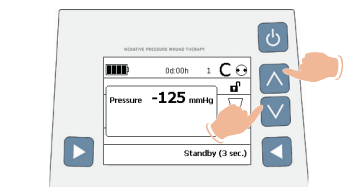

2. Press selection buttons up and down simultaneously.

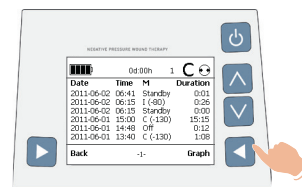

3. Press "Graph" to view graph.

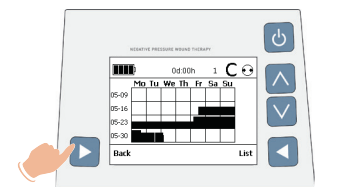


4. Press "Back" to return to main menu.

Air leak indicator

- System is air tight
- Air leak detected
- Significant air leak → Alarm within 2 min
Operation stops when not resolved (7 min)

*High air leakage mode is recommended when X-large dressing is used. Refer to section "Select air leakage volume" to change between Standard and High modes.

Display symbols

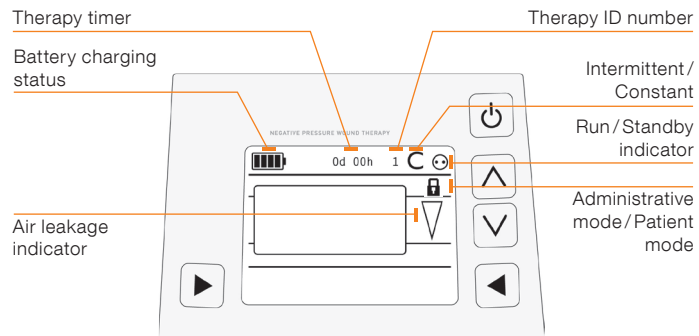

Alarm table

Fault number	Problem description on the display	Troubleshooting on the display	Pressure	Remarks/potential cause of fault
301	Air leak in system	Check dressing for air leakage and if canister is properly inserted. Consult IFU for further instructions.	✓	Dressing: – Check dressing for air leakage. Press firmly around the edges of the dressing, around the drain tubing or on the Invia FitPad. Connectors: – Ensure that the wound dressing tubing is connected properly to the canister tubing. Canister: – Ensure that the canister tubing is inserted straight into the pump. – Ensure that the canister is properly inserted, release the canister and reposition. – Ensure that the orange O-ring/gasket, placed beside the canister tubing on the pump is not missing. Additional O-ring is available via Medela customer service.
302	System clogged	Check that tubing is clear, not kinked and clamp open. Check if canister is full. Consult IFU for further instructions.	✓	Tubing: – Ensure that the tubing is not twisted, kinked or clamped. Canister: – If dressing tubing is clogged, change the dressing. – If canister is full or filter clogged, replace canister.
305	Battery empty	Charge battery	✗	Recharge the battery either by placing the Invia Liberty pump in the Docking Station or plug in the charger to the electrical outlet port on the pump. Then press "On" to restart the therapy. Remaining time of battery is 15 minutes.
306	Canister full	Change canister	✓	Change the canister, see chapter "Change Invia Liberty Canister and Invia Liberty Tubing".
311	Selftest failed	Snap the canister out and in again	✗	Canister: – Release the canister and reposition. Tubing: – Ensure that the tubing is not twisted, kinked or clamped.
312	Pump in Standby Leak in system	Check dressing for air leakage and if canister is properly inserted. Consult IFU for further instructions.	✗	Occurs when alarm 301 was pending for 5 minutes. For troubleshooting, refer to instructions for Alarm 301. Restart the pump by pressing "On" [On].
313	Filter clogged	Change canister	✗	Change canister, see chapter "Change Invia Liberty Canister and Invia Liberty Tubing". To restart the therapy after canister has been changed, press right selection button "On" [On].
315	Acceptable internal temperature exceeded	Switch pump off and on. If problem persists, contact your Health Care representative.	✗	Appears when warning 406 was pending for 30 min. Cool Invia Liberty down.


Warnings and alarms

Example

! Warning




!!! Alarm

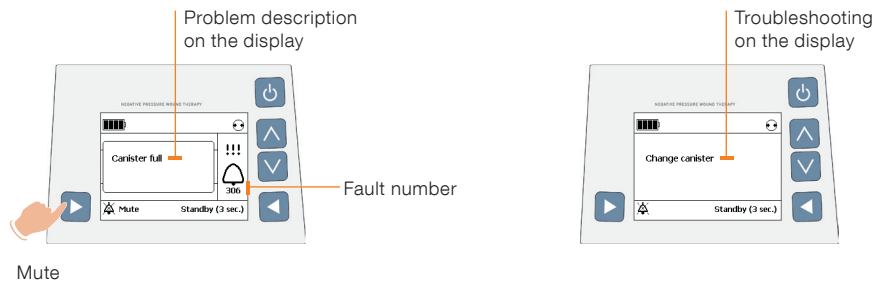

	Fault number	Problem description on the display	Troubleshooting on the display	Pressure	Remarks/potential cause of fault
Warning	401			✓	Recharge the battery either by placing the Invia Liberty pump in the docking station or plug in the charger to the electrical outlet port on the pump. Remaining time of battery is approximately 30 minutes.
	402				Unplug USB cable.
	405			✗	If the pump is in Standby mode for more than 5 minutes, an alarm will go off. To continue therapy press "On" [▶] or switch off the pump by pressing [⏻ > 3 seconds].
	406			✓	Cool Invia Liberty down.
Internal fault				✗	Restart the pump. If internal fault remains, turn off by pressing [⏻ > 3 sec.] and contact Medela Customer Service.

Invia Liberty pump and accessories

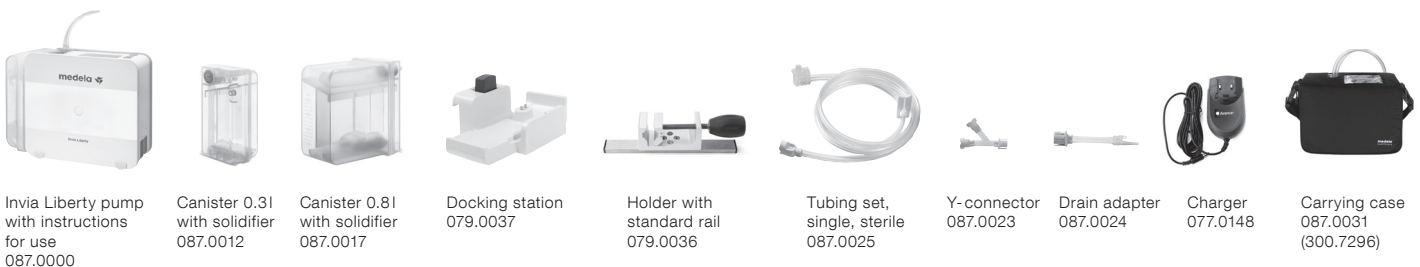

Invia Liberty pump with instructions for use
087.0000

Canister 0.31 with solidifier
087.0012

Canister 0.81 with solidifier
087.0017

Docking station
079.0037

Holder with standard rail
079.0036

Tubing set, single, sterile
087.0025

Y-connector
087.0023

Drain adapter
087.0024

Charger
077.0148

Carrying case
087.0031 (300.7296)

CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician.
Medela, Invia, Invia Liberty and Invia FitPad are registered trademarks of Medela AG Switzerland.

Medela AG
Lättichstrasse 4b
6341 Baar, Switzerland
www.medela.com

USA
Medela LLC
1101 Corporate Drive
McHenry, IL 60050
USA
Phone +1 877 735 1626
Fax +1 815 307 8942
info-healthcare@medela.com
www.medela-healthcare.us

Canada
Medela Canada Inc.
4160 Sladerview Cres., Unit #8
Mississauga, Ontario, L5L 0A1
Canada
Phone +1 800 435 8316
Fax +1 905 608 8720
info@medela.ca
www.medela.ca