

NEGATIVE PRESSURE
WOUND THERAPY

Avance® Negative Pressure Wound Therapy Quick Reference Guide




 Avance® Flex Pump


APRIA HEALTHCARE®

REVISION 2018-04

Before initiating NPWT treatment, read the Instructions For Use, indications, contraindications, warnings, precautions and safety instructions.

Display

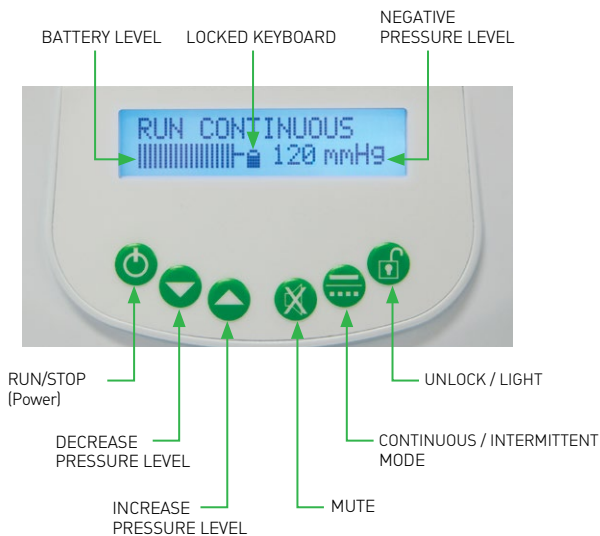

Power On / STANDBY mode



On: Press and release RUN/STOP (Power) button

The pump is now in STANDBY mode.



When turning the pump on the first time, the default settings will be -120 mmHg and continuous therapy. Ensure the pressure setting is adjusted to the pressure level ordered by the prescribing physician prior to initiation of therapy.

Change Pressure level




When the buttons are unlocked, pressures can be modified in STANDBY and RUN mode, in 5 mmHg increments between -60 mmHg and -180 mmHg using the up and down arrow buttons.

Change between Continuous and Intermittent



Switch between therapy modes: First unlock buttons if locked. Press the CONTINUOUS/INTERMITTENT mode button until the pump beeps to confirm change of mode.

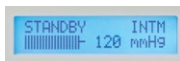

When in INTERMITTENT therapy mode, the pump will provide negative pressure therapy for periods of 5 minutes followed by a 2 minute rest period with no negative pressure provided.

Start the Therapy



START the therapy: Press the RUN/STOP (Power) button again.



If not adjusted, the pump will begin operation at the default pressure of -120 mmHg, and CONTINUOUS therapy mode. The selected negative pressure level is shown on the display screen. Ensure the pressure setting is set to the pressure level ordered by the prescribing physician prior to initiation of therapy.

Unlock the buttons



After 1 minute in RUN mode, the locked key function is activated. This is shown with a padlock sign.

Press the LOCK/UNLOCK key for 3 seconds to unlock the buttons.

The pump will lock again 1 minute after the last button operation.

Stop therapy and power off the system



STOP therapy: First unlock buttons if locked. Press the RUN/STOP (Power) button until the unit beeps. The pump will power off.

Power off the system from STANDBY mode



If in STANDBY mode, the system will self power off when inactive for 1 minute when on battery only operation.

You can force power off from STANDBY mode by pressing the MUTE button until the unit beeps.

Illumination of the screen



Pressing the MUTE button will illuminate the display for 10 seconds.

When the AC (wall) power supply is connected, the display will stay illuminated.

Mute the warnings



Press the MUTE button to silence the warning indicator and to clear the message from the display screen.

Warnings

Warning	Display / Warning	System Operation	Remedial Action
No Canister	'No Canister' / Warning beeps	Will not activate RUN mode from STANDBY mode Reverts to STANDBY mode if canister is removed when in RUN mode	Attach a new canister (see next section) If the warning continues, contact the helpline (see number on last page)
Canister Full	'Canister Full' / Warning beeps until muted Display re-occurs after 10 minutes until remedied	Will not activate RUN mode	Replace the canister
Leak Detected	'Leak Detected' / Warning beeps, display flashes	RUN mode continues Warning indicator is mutable but re-occurs if no action taken	Identify the location of leak Seal the source of the air leak Use no more than two layers of Avance® Film
Tilt Warning	'Tilt WARNING' / Warning beeps after 30 seconds	Continues in RUN mode	Put the pump back in the upright position
Blockage	Blockage / Warning beeps, display flashes	Continues in RUN mode Warning is mutable but re-occurs if no action is taken	Ensure tubing clamps are open Resolve any blockages in the tubing If the warning repeats, place unit at or below wound site and contact technical support

Change the canister and tubing



To change the canister make sure the pump is powered off (if still running, press and hold the UNLOCK button, followed by the RUN/STOP [Power] button).



Clamp the canister tubing and remove by turning counter-clockwise and lifting it out of the tubing port.



Remove the sealing plug from its location on the canister. Use the plug to seal the tubing port.



Rotate the locking knob 1/4, turn counter-clockwise and remove the canister. Dispose of the used canister according to local clinical waste regulations.



If continuing NPWT, attach a new canister. Remove the styrofoam pin guard on new canister prior to attaching. Attach the canister to the flat face of the pump by matching up the rear location pegs and rotating the locking knob 1/4 turn clockwise to secure. Ensure the canister is correctly located and secured. Otherwise a NO CANISTER message will appear and the pump will not operate.



Attach the tubing to the canister. Push down gently and twist clockwise to lock. Unclamp the canister tubing.



Power on the pump by pressing the RUN/STOP [Power] button. The pump is now in STANDBY mode. Ensure the pressure is set to the prescribed level by pushing the up or down arrows.



Press the RUN/STOP [Power] button again to start the therapy.

Notes



APRIA HEALTHCARE®

Visit us at [apria.com](https://www.apria.com)

Apria Healthcare LLC complies with applicable Federal civil rights laws and does not discriminate on the basis of race, color, national origin, age, disability, or sex.

Apria Healthcare LLC cumple con las leyes federales de derechos civiles aplicables y no discrimina por motivos de raza, color, nacionalidad, edad, discapacidad o sexo.

Apria Healthcare LLC 遵守適用的聯邦民權法律規定，不因種族、膚色、民族血統、年齡、殘障或性別而歧視任何人。

The Mölnlycke Health Care, Avance, CFM Technology, ViewPad, Mepitel, Mepiseal and Safetac trademarks, names and logo types are registered globally to one or more of the Mölnlycke Health Care Group of Companies. Distributed by Apria Healthcare LLC. © 2018 Mölnlycke Health Care AB. All rights reserved.

Manufacturer: Talley Group Limited, Premier Way, Abbey Park Industrial Estate, Romsey, Hampshire S051 9DQ, England.